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ITEM: 871125212370

Shower stool

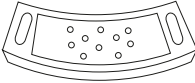
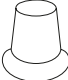
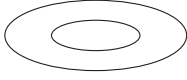
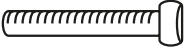
Dear customer, Your new shower stool offers you optimum functionality and comfort; it will make it easier for you to take a shower. Please read these instructions carefully before you assemble and use this product. Keep the instructions for future reference. The manufacturer and importer cannot accept any liability for the incorrect use of this product or failure to observe these instructions!

Intended use

The shower stool is only to be used as a seat to support persons in the shower. It is designed for indoor and domestic use.

Note: not for use in the rehabilitation or medical treatment of persons.

Contents of package

1 x A	1 x B	1 x C	4 x D
			
Support seat	Seat tube	Seat tube	Leg tube
4 x E	5 x F	5 x G	
			
Rubber foot	Washer	Round/ headed screw (M6 x 35 mm)	

Safety instructions

- Keep the packaging material out of the reach of children (suffocation risk).
- Keep the shower stool out of the reach of children. Children could attempt to climb on or pull themselves up by the chair and fall over with chair.
- Make sure that another person is available who can help you to sit down or stand up if necessary.
- Do not carry the chair by the recessed grips in the seat when a person is sitting on the chair. The sides of the seat could break off or bend and cause injury.
- This shower stool can be used by persons with a maximum weight of 150 kg.
- Inspect the shower stool before use and ensure that it is standing securely. Make sure that there is a nonslip underlay on the shower floor and outside the shower. You could slip on a wet floor and injure yourself.
- Inspect the shower stool once a month to make sure that it is stable, the rivets and all joints are secure and that the screws are tight. Tighten if necessary.
- Do not use corrosive and abrasive cleaners.
- Do not modify the product or try to repair it.
- We cannot accept any responsibility for damage to the shower floor.

- Do not stand on the shower chair. Sit in the chair properly to avoid injury.
- Do not use this product if you can see or think that it is damaged.
- Smooth or damp surfaces can have an adverse effect on the slip resistance.
- Do not lift with persons on it.

Assembly

Fig. 1 Push the rubber foot (E) onto the leg tube (D). See Fig. 1. Repeat for the other leg tubes.

Fig. 2 Fit the leg tubes with the rubber feet onto the seat tubes (B) and (C). See fig. 2

Fig. 3 Fit a washer (F) to each screw (G) and (H) and fasten the seat tubes (B) and (C) to the base of the seat (A). Tighten the screws. See Fig. 3

Fig. 4 Press the pushbuttons into the leg tubes (D) and slide the legs to the desired height. When released, the pushbuttons must project through the holes. Set all the leg tubes in the same height to avoid the risk of chair tipping. Make sure that the chair is stable by turn the rubber feet (E) until they are resting firmly on the shower floor. See fig. 4

Technical specifications

Dimensions (LxWxH): ca. 51x43x57cm

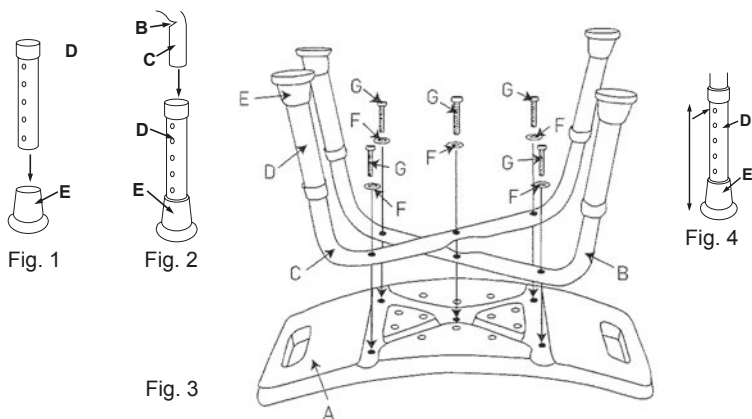
Seat surface: ca. 51x31.5cm

Adjustable height: ca. 39-57cm

Chair weight: ca. 1,9 kg

Max. load: 150 kg

Recommended period of use - 3 years

Cleaning

Do not use any caustic or abrasive cleaning agents or sponges. These can damage the surface.

1. Wipe off the shower stool with a damp cloth and a small amount of mild detergent if necessary.
2. Then dry the seat surface completely before using it again.

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